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Instructions for Use Diagnostic Light

Intended Purpose

The diagnostic light is designed to illuminate the mouth, throat, and skin for diagnosis purposes.

Use

The diagnostic light must be used by qualified personnel only.

Warning

When examining the mouth and throat, make sure that the diagnostic lamp cannot fall into the oral cavity. Danger of choking! The diagnostic lamp may not be used for examining the eyes!

Note

Check the diagnostic light for damage before use. Do not use if damage is visible. Visually inspect the bulb before use. Avoid direct contact with the tongue and mucous membrane. Do not place the diagnostic light in liquids.

Storage

Store in a dry place between 10 - 30°C.

Cleaning/Disinfection

Clean the surface of the diagnostic light with a damp or dry cloth.

Disposal

The product must not be disposed of with household waste. It must be disposed of at a municipal collection point for electrical and electronic equipment.

Warranty / Shelf Life

This product comes with a two year warranty commencing on the date of purchase when handled properly and in compliance with the instructions for use. This warranty extends to all parts with the exception of the bulb and battery. The diagnostic light has a shelf life of 5 years.

Note for users and patients

Please report serious incidents that have occurred in connection with this product to the manufacturer and the competent authority of the Member State in which the user and/or patient is located.

Symbol Explanation

	The CE mark confirms the compliance to applicable European (EU) regulation		Catalogue number
	Medical Device		Batch code/ Lot number
	Separate disposal of electrical and electronic waste. Used batteries must be disposed of separately		Manufacturer
	Read instructions for use		