SPECIFICATIONS ROMED STERILE SYRINGES FOR SINGLE USE LATEX FREE, 100ML

REF: 3SYR100ML-LT



Applicable harmonized standard: EN ISO 7886-1:1997 Medical device conform MDD 93/42/EEC

Name of the product:

ROMED sterile syringes for single use

Product characteristics

Barrel Polypropylene (PP)
Plunger Polypropylene (PP)
Piston Isoprene, Latex-Free

Volume: 100 ml

Specifications:

Tip type	Catheter tip	Length of barrel	167.4 mm
Tip position	Concentric	Length of plunger	155 mm
Inner diameter	35.42 mm	Size of rubber piston	10.06 mm
Graduation	5 ml	Overall length	200.35 mm

Product is phthalates and latex-free

Syringes are not resistant to UV radiation.

Tooling parameters for raw materials:

Limit for extractable metals : Total \leq 5 μ g/ml, CD \leq 0.1 μ g/ml

 $\label{eq:limits} \mbox{Limits for acidity or alkalinity} \qquad \qquad : \quad \triangle \mbox{ pH} \leq 1 \mbox{ pH}$

Oxidized material : $KMnO_4$ consumption ≤ 0.5 ml

Method of sterilization : EO gas

Non-toxic, non-pyrogenic

The product is injection moulded with PP a three-piece structure (barrel, plunger, piston). The luer nozzle is situated centrally.

If the single package is opened before use, discard it immediately and do not use it. Take care not to contaminate the product during unpacking. Assemble the injector with the syringe, suck a given amount of desired infusion, remove air and inject. The product is limited for single use and after use it must be discarded immediately.

The product remains sterile for a period of 5 years after sterilization, provided that it is kept under specified storage conditions. Do not use after expiry date.

Inspection item	Technical requirements		
Tightness of injector body	No liquid leakage under 300 kPa pressure and 1.0 N action force for 30 seconds		
Air tightness of suction	No air leakage under 88 kPa negative pressure for 60 ± 5 seconds, no separation of plunger		
Error in capacity	Max. error of capacity \pm 5% when above $\frac{1}{2}$ main nominal mark line		
Tightness of conical head	No water dropping under 300 kPa water pressure for 30 seconds, no continuous air bubble during suction		
Separation force of conical head	No separation of the under-test inner conical head under 25 N reversed force		
Sleeve length	At least 10% more than the length for the nominal capacity		
Handle interval	≥ 9.0 mm		
Requirements on conical head	≥ 1.2 mm, coaxial of sleeve with head-middle type injector		
Dimensions of conical head	conforming to ISO 7886-1:1993		
Flanging	no turning over in a place inclined at 10° with the level		
Scale	Length \geq 27 mm, middle deviation type injector keeping at opposite side of conical head		
Graduation mark	Graduation 0.2 mm or 0.1 mm, evenly separated, flushing of marks for same volume, length of secondary mark ½ of that of main mark		
Position of zero mark	Aligning with base line of piston, error ≤ ¼ graduation interval		
Appearance	Free of flash, burr, sagging, defects, clean surface free of foreign matters, transparent sleeve, base line visible		
Slippage	Good slippage, meeting requirements on pull or push force		
Residual amount	≤ 0.07 ml		

Packing

3SYR100ML-LT: Per piece in sterile blister, 25 pcs in an inner box, 4 inner boxes in an export carton