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Cuticell® Classic

Paraffingaze

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Cuticell® Classic

Paraffin gauze dressing

Cuticell® Classic is a sterile, non-medicated, paraffin gauze dressing with open weave.

Properties

Due to its paraffin coating, Cuticell® Classic is soothing and low-adherent. The open weave allows exudate to pass through into an absorbent secondary dressing. The gauze has interlocking threads which minimise fraying when the dressing is cut to shape.

Indications

Cuticell® Classic is used as a primary wound contact layer in the treatment of exuding wounds such as

- Minor burns and scalds
- Donor and recipient skin graft sites
- Skin loss wounds
- Lacerations and abrasions
- Leg ulcers

Instructions for use

Prior to application, the wound should be cleaned as usual. Remove Cuticell® Classic from the peel pack and cut it to the size of the wound if necessary. To absorb wound exudate, a sterile absorbent pad can be placed over Cuticell® Classic. For fixation, bandages or adhesive material can be used. Cuticell® Classic should be removed at the latest as soon as the paraffin starts to liquefy.

Contraindications

There are no known contraindications for Cuticell® Classic.

Warnings and Precautions

Cuticell® Classic is packaged for single use. Do not re-use or re-sterilise as there is a risk of transmission of body fluids or contaminated tissue between patients. Do not use if pouch is open or damaged. Using Cuticell® classic in multiple layers or on heavily exuding wounds may cause occlusion. Impregnated gauze dressings may stick to the wound bed of dry wounds.

Storage

Cuticell® Classic should be stored in dry conditions below 50 °C/122 °F.

Assortment

Cuticell® Classic is available in different dimensions and pack sizes:

REF No	Sizes	Dressings / pack
72538-00	5 x 5 cm	5
72538-01	5 x 5 cm	50
72538-02	10 x 10 cm	10
72538-03	10 x 10 cm	100
72538-04	10 x 40 cm	10
72538-05	10 x 10 cm (dispenser)	36 (not individually wrapped)
72538-06	7 m x 10 cm (dispenser)	1
72538-07	15 cm x 2 m	12

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© = registered trademark
**BSN medical GmbH**