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Patient restraint

Purpose:

The restraint is used to secure a patient's limbs.

Product description:

The patient restraint is a padded fixation and can also be used over a longer period of time. An additional integrated Velcro fastener allows variable adjustment to the respective extremity.

- Universal fit thanks to the adjustable Velcro closure
- Stable restraint eyelet

Instructions:

The Patient restraint is placed tightly around the Limbs and closed with the Velcro strap. Do not place the patient holder too tightly, otherwise there is a risk of blood stasis. The eyelet attached to the patient holder can be used with an appropriate retaining strap for fixation (e.g. to a bed frame).

Warning:

Never put the restraint on too tightly, as this could cause the blood to congest. Only to be used by qualified personnel. The patient restraints are not to be used for psychiatric purposes. Disposable.

Legal note:

In the event of any serious incidents involving this product, please inform the manufacturer and the responsible authority in the member state where the patient and/or user is located.

Disposal:

The product must be disposed of in accordance with local regulations.