SPECIFICATIONS ROMED PILL CUTTER

REF: PC-480



Medical device conform MDD 93/42/EEC

Pill cutter can be used to cut pill exactly in half for easier swallowing or to ingest only half the dose of the medication.

It is safe and easy to use.

With locking button, protect user from potential injury from sharp stainless steel blade.

Materials: Plastic, stainless steel

Length: 8.5cm
Width: 3.5cm
Height: 2.8cm
Weight: 30g

DIRECTIONS FOR USE

- 1. To open, simply press on the locking bar.
- 2. While pushing the locking bar in, lift up on cover.
- 3. Place pill into the "V" shaped holder.
- 4. Close cover to split pill.
- 5. Repeat opening process and remove pill halves.
- 6. Close cover and store in closed position.

Packing: Each in a box, 24 pieces in an inner box, 480 pieces in an export carton.