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Product description:

RUDASPRAY® is an adhesive spray for use on bandages.

Contains: propellants (propane, butane), special benzine, acetone, binding agents, DINP, dye (blue)

Accessory for medical devices.

Intended use and areas of application:

RUDASPRAY® improves the fixation of individual layers of bandages, dressings and tapes. RUDASPRAY® is applied by medical professionals.

Application instructions:

Wear appropriate protective gloves. Suitable gloves are chemical protection gloves that have been tested in accordance with EN 374. Work in well-ventilated rooms, free from draught. Briefly spray the bandage 1–2 times at a distance of 20–25 cm. Wait for 30 seconds and press on a further layer. Do not inhale.

Contra-indications:

Do not apply the product in the case of a known allergy to the material.

Warnings and precautions:

- Do not inhale.
- Do not use at the eyes or on the skin or mucous membranes!
- After contact with skin, wash with plenty of water. Repeated skin contact may cause brittlel and chapped skin.
- Remove stains on clothes or skin with alcohol. Do not use solvents.
- Observe the hazard warnings mentioned below and on the product.
- Should serious incidents occur in relation to the device, they are, according to MDR (EU) 2017/745, to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Storage and transport:

Store in a dry and dust-free environment, out of direct sunshine. Do not expose to temperatures exceeding 50°C.

Waste disposal:

In accordance with the requirements of the applicable local legislation and standards. Dispose of cans with residual contents at hazardous or special waste collection points.

	
---	---

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